## MONO-DOX™

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*Mono-Dox™ us. vet.*

#### **Composition :**

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*Mono-Dox<sup>®</sup> us. vet.*

**Composition :**

Monofilament sterile, absorbable suture for veterinary use, made from polyester and poly(p-dioxanone)polymer.

**Marketing authorisation holder and responsible for the batch release :**

Spri Docteur Michel Frère Route de la Belle-Vue, 17 B-6833 UCIMONT Belgique (tel : 00 32 (0)61 46 73 45 – fax : 00 32 (0)61 46 86 40)- [info@michelfrere.com](mailto:info@michelfrere.com) - [www.michelfrere.com](http://www.michelfrere.com)

**Target species :**

All species.

**Indications :**

Polydioxanone is indicated for use in general soft tissue approximation and ligation.

**Recommendation for appropriate use :**

- Polydioxanone retains approximately 70% of the original strength at two weeks post-use; 50% at 4 weeks, and 25% at 2 weeks. Absorption is essentially complete at 6 months.
- Users should be perfectly familiar with surgical procedures and techniques involving absorbable sutures before using for wound closure as the risk for wound dehiscence may vary with the site of application.
- Avoid damage when handling. Do not grip the wire with surgical instruments except the free end of the suture when making ties.
- Sterility is not guaranteed if package is opened or damaged.
- Neither re-use, nor re-sterilise used suture.

**Contraindication :**

The suture absorbable and will not be used for prolonged approximation of soft tissues.

**Storage particulars :**

Before opening :

Mono-Dox<sup>TM</sup> will be stored below 25 °C.

After opening :

See Particular warnings. Opened packs will be stored in a dry and cool place. Sutures will be used within 120 days after opening of the packs, respecting strictly the recommended storage conditions.

**Particular warning :**

Polydioxanone sutures are very sensitive to humidity, and even more, to warmth. For that reason, it is of paramount importance to store opened packs at the recommended storage conditions and to use the content within 120 days after opening.

**Last update :**

**September 2005**